

USER MANUAL



Set up, lubrication and pack down
instructions of the Canine Oesophageal
Feeding Tube Model



Oesophageal Feeding Tube

Revision A: Oesophageal Feeding Tube	Product SKU: C400-000	Last Updated: 30 August 2023
--	--------------------------	---------------------------------

Parts List

Equipment Required:

- 1x Dog Body
- 2x Surgical Block

Spare Parts Available

1. Surgical Block
2. Table Bracket

For all spare parts, repairs, and all other inquiries, please get in touch via www.holsim.co.nz.

The Parts



The Set Up



1 Position dog

- Position the dog so that the surgical block cavity is facing upwards.



2 Insert surgical block


- Place the surgical block in the main dog body with the Velcro facing downwards.
- Make sure the skin from the main dog body and the surgical block line up properly.



3 Mount model to table - if required.

- Use the mounting points and screws at the rear of the model to attach an appropriate bracket to the model to hold it in place.
- Please note – mounting brackets is not supplied, if a mounting bracket is needed contact us: Holsim Animal Care and Education

The Lubrication

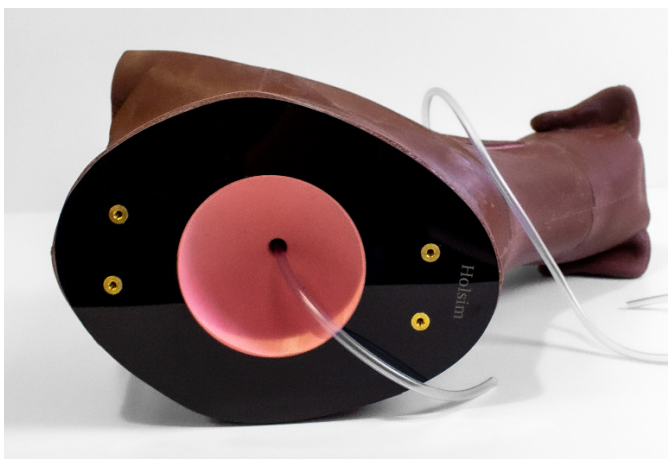

1

Add lubricant

- Add lubricant to any tubes or objects being inserted into the model.
- This will prevent any tears or punctures from occurring internally.



2

Carry on with operation

- Perform the operation as you normally would.

Pack Down & Storage



1 Remove surgical block

- Removing surgical block by gently pulling the block out by hand.
- The surgical block is held in place via Velcro.



2 Clean all components

- Spray down all components with anti-bacterial spray and wipe dry.
- Make sure all parts are completely dry before storing.



3 Safely store away

- Make sure the model is free of any dust, dirt, and moisture.
- Store the model in a clean and dry environment to prevent mould and product deterioration.



34D somerset Street, Frankton ,
Hamilton, 3204
New Zealand